



Prescription Preferences Update

Setting up Prescription Preferences

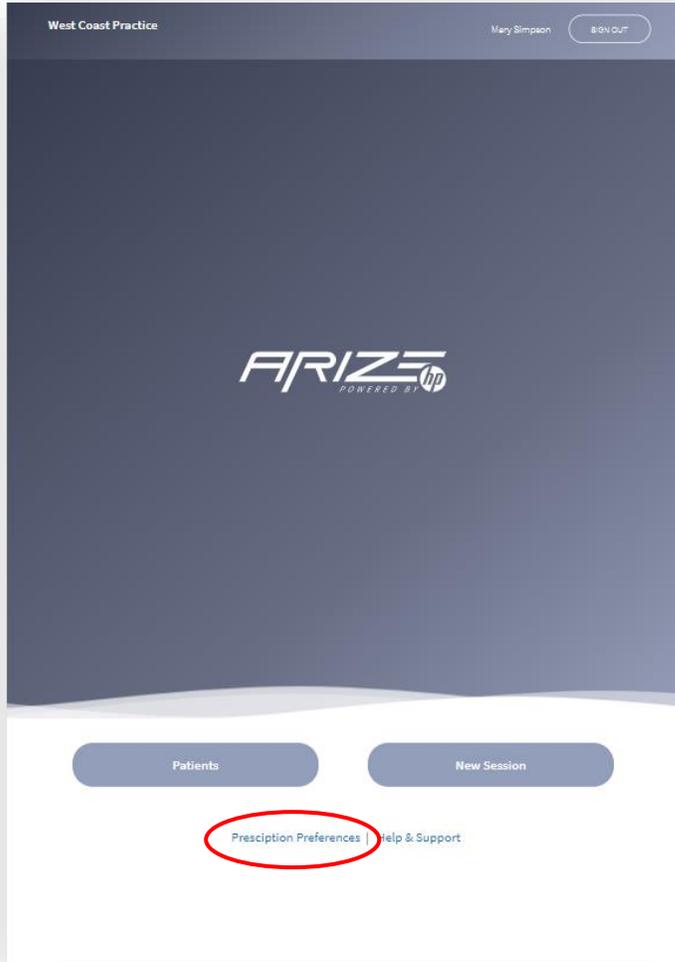

- A Setting up the Prescription Preference remains the same.
- B From the Home screen, select Prescription Preferences at the bottom of the screen.

Modifying Default Styles

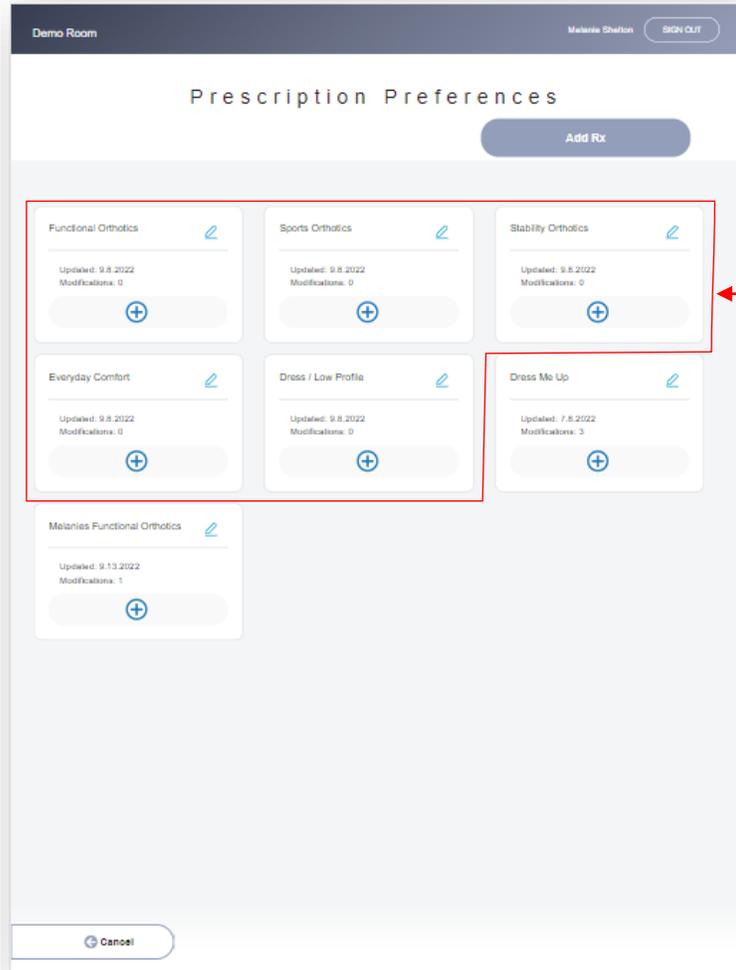

Default styles

A You can now view the Arize default styles in the Prescription Preferences setup screen

- Functional Orthotics
- Sports Orthotics
- Stability Orthotics
- Everyday Comfort
- Dress/Low Profile

B You can modify the default styles and save them as a new style.




You cannot rename or delete an Arize default style


How to Edit and Save an Arize default style




On the Home screen, select the Preferred Prescription link

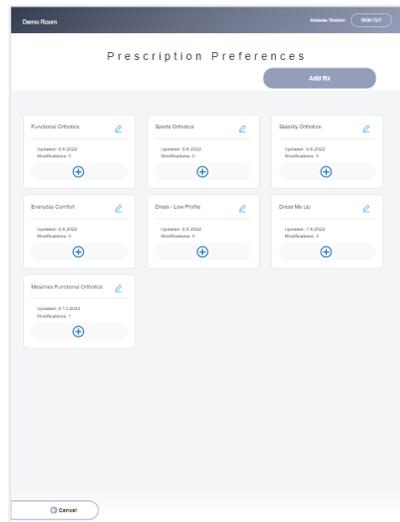

You can either create a new style or duplicate and modify an Arize Default Style

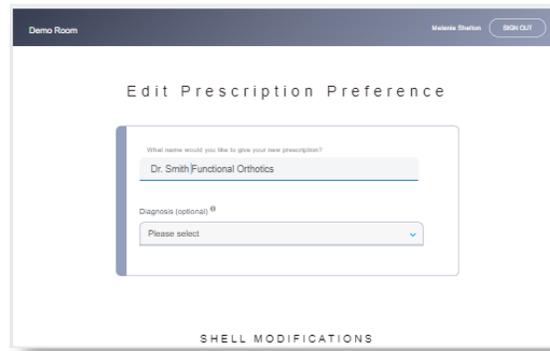

Give the style a new name

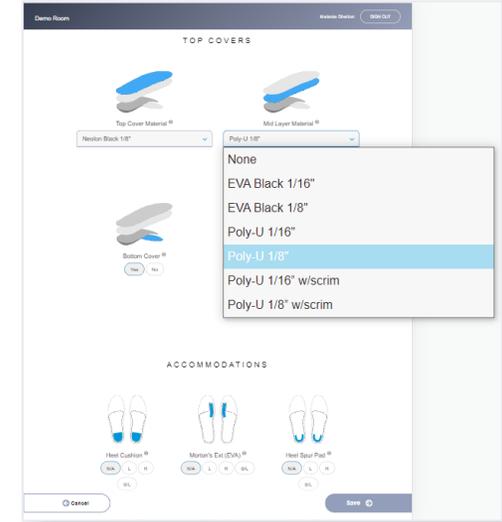


Change any settings on your new default style


Click save when done

How to select your Prescription Preferences

A Step 1 – Follow your normal new patient flow

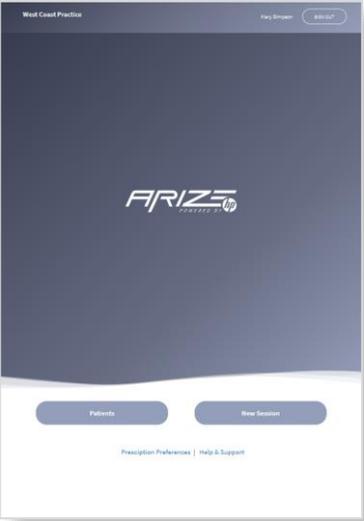


Start a new Session

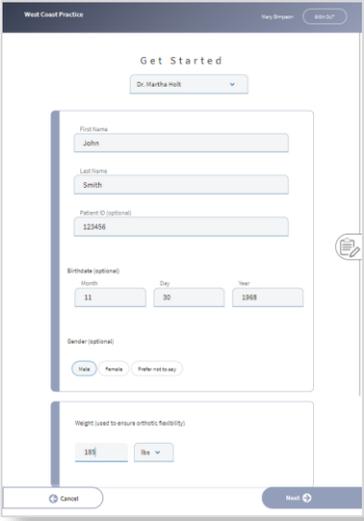


Enter the patient's information then click on Next



Scan the patient's feet then click on Next

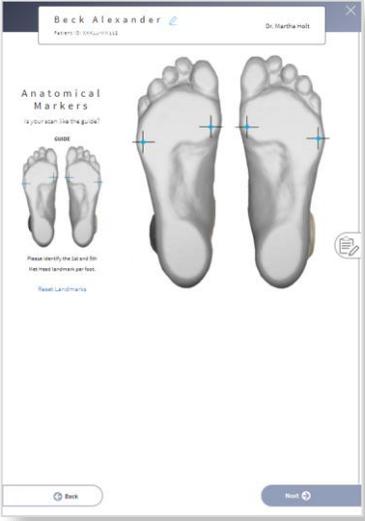



Select the anatomical markers then click on Next

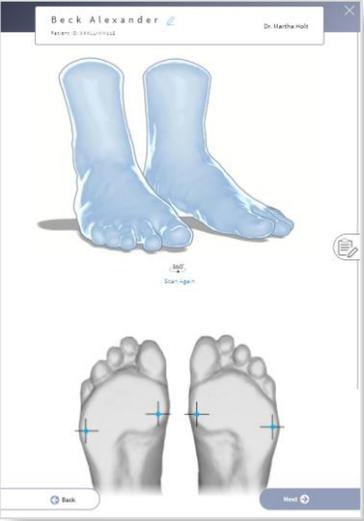


Review the scans and click Next

The New Orthotic Style page

When you reach the orthotics style page you will notice the design has changed. You will now see the screen on the right.

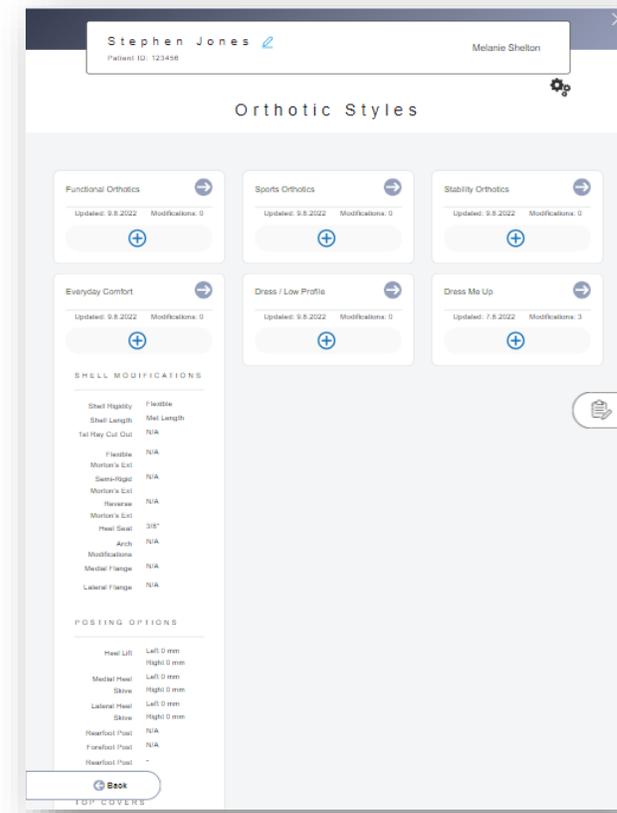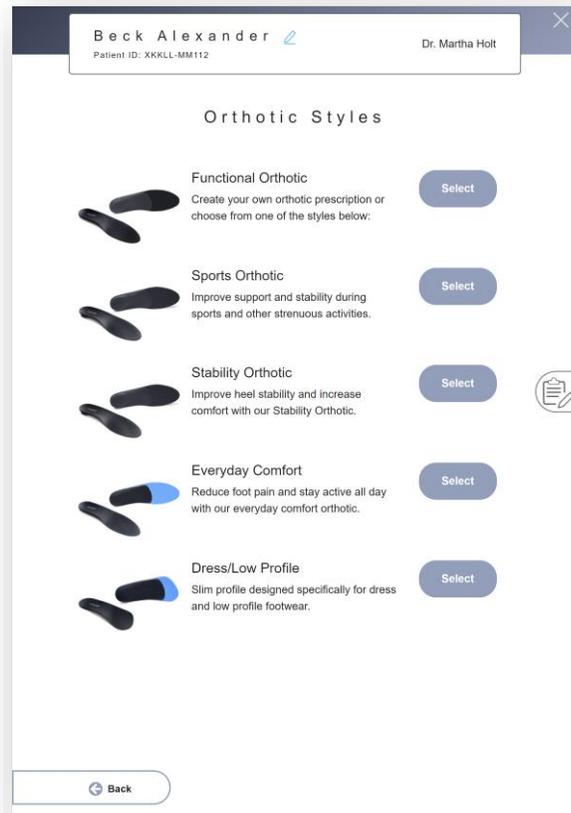

Step 2 – Select your Style

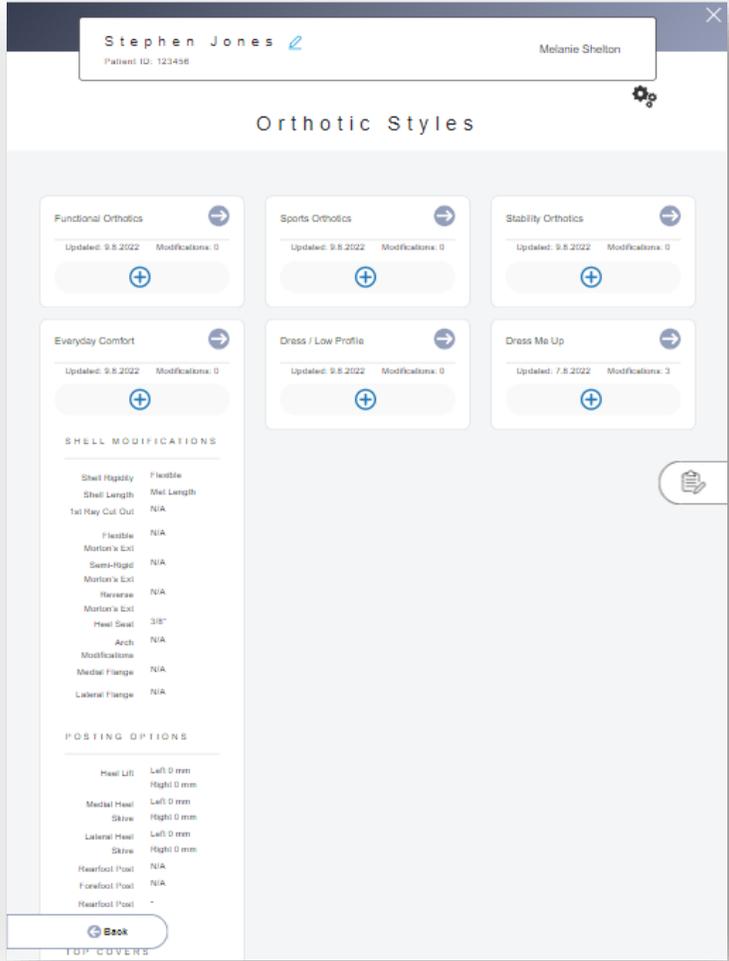

- A** Arize Default styles and the doctor's Prescription Preferences are viewable on the Orthotic Styles page
- B** Click on the **+** icon to see the modification summary of that style
- C** Click on the **→** to select the Style you want to use for the patient's prescription
- D** Note: When the app is updated, existing Prescription Preferences will show up on this page. You will not need to recreate them